




SILVERNEST

N95 PROTECTIVE MASK

HOW TO WEAR
WEARING METHOD



- 01** Perform Hand Hygiene Before Putting on A Mask.
Position The Mask Under Your Chin With The Nosepiece Up.
The Ear Straps Should Be Pulled Using Each Hand.
- 02** Face The Mask Against The Chin First. Pull The Straps Back To The Ear.
Hang It on Until Feel Comfortable. Position The Mask Low on Your Nose.
- 03** Using Both Hand, Mold The Nosepiece To The Shape of Your Nose By
Pushing Downward And Outward. While Moving Your Fingertips Down
Both Sides of The Nosepiece.
- 04** Perform A Fit Check By Placing Both Hands Completely over The Mask.
Be Careful Not To Disturb The Position. And Exhale Sharply. If Air Leaks,
Adjust The Nosepiece As Described in Step 2 And 3.


SILVERNEST

N95 PROTECTIVE MASK

PFE \geq 95%
FILTRATION

Face Mask

5 Layers

- High quality material
- Good quality elastic
- Adjustable nose clip
- Purifying The Air Layer By Layer
- 5 Layers of Filtering Multiply Protection
- Foldable mask

25
PIECES

N95 PROTECTIVE MASK

Face Mask
5 Layers

25
PIECES



SILVERNEST

N95 PROTECTIVE MASK

PFE $\geq 95\%$
FILTRATION

Face Mask
5 Layers

- High quality material
- Good quality elastic
- Adjustable nose clip
- Purifying The Air Layer By Layer
- 5 Layers of Filtering Multiply Protection
- Foldable mask

25
PIECES